

# ProGuard

HEARING PROTECTION

**INSTRUCTIONS: Please Read carefully before using your ProGuard IFB Custom Moulds**

1. Before fitting any custom fit ProGuard products, make sure your hands are clean.
2. Reach around your head and pull up and back on your outer ear. This straightens out the ear canal, making way for a perfect fit when the custom mould is inserted. N.B. The orientation of your ear canal is backward and upward, in the general direction of the top centre part of your head .
3. ProGuard IFB are supplied with a curly acoustic tube and collar clip; most standard earphone drivers will simply snap into the tube fitting. If you need more information or require earphone drivers or other fittings please contact us and we can advise the most appropriate driver and cable combinations.
4. The acoustic tube fitted to the earpiece is oriented to fit over the top of the ear.
5. Earwax may build up and can block the audio output port; carefully use the supplied cleaning brush and wire-loop to clear the wax; earwax build up is nothing to worry about and is perfectly normal.

*Please Note: Your ProGuards should be cleaned often in warm mild soapy water and dried thoroughly before use.*

ProGuard UK,

Market by **Sensorcom Ltd**,  
4 – 6 Thayers Farm Road,  
Beckenham  
BR3 4LZ  
[www.proguarduk.co.uk](http://www.proguarduk.co.uk)  
+44 870 901 6070